

DORSAL SLEEP APPLIANCE


PATIENT INFORMATION

Operating Instructions

The appliance should be placed in mouth prior to sleeping. The appliance should remain in for the entire sleep period.


- **Cleaning** – Brush with toothbrush and toothpaste before each use
- **Storage** – Store dry in provided case when not in use.
- **Fit Adjustment** – All adjustments should be done by the prescribing dentist or physician.
- **Product Life & Replacement** – Expect a 2-3 year product life with the appliance.

Appliance replacement signs are; cracks in the acrylic and failure of the device to maintain retention in the patient's mouth. If one of these problems occurs please return the appliance to your doctor/dentist for evaluation and potential replacement.

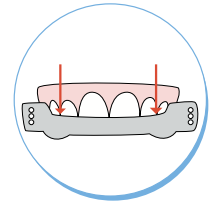

TO PROPERLY REMOVE

Using both index fingers, pull down in the front area of the upper appliance.

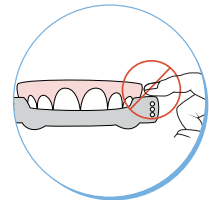

TO AVOID BREAKAGE

DO NOT pull down the back of the appliance near the screws with finger pressure.