

Digital Dentistry is not just the future of dentistry, but the present. We at Pan-Am embrace technology, and aim to be up to date with the latest products. We accept all STL files from intraoral scanners, and are committed to providing the most advanced and up-to-date digital services you need to move your practice forward with confidence. Get iTero Scanners at your location, and a \$500 monthly lab bill discount!

## CEREC

### Sending A File

- 1.) Create an account on [cerec-connect.com](http://cerec-connect.com) and follow the prompts to activate an account.
- 2.) Log into your account and select Pan-Am Dental Lab from the list of available labs.
- 3.) Add Pan-Am Dental Lab to your list of "my favorite Laboratories"
- 4.) Complete your scan and online Rx form. Choose Model-Less if you do not want a model.
- 5.) Select Pan-Am Dental to be the file recipient.



## TRIOS

### Sending A File

- 1.) Go Online and register with 3Shape Communicate and send Connection Request to Pan-Am Dental Lab.
- 2.) Pan-Am will respond that it accepts the request to send files.
- 3.) Complete the intraoral scan and Trios Rx form. In the notes section, be specific about what materials you want to use and any other information.
- 4.) Select Pan-Am Dental Lab and Send File

## iTERO

### Sending A File

- 1.) Sign into the Cadent iTero account online and add Pan-Am Dental Lab to your scanner menu.  
If you have trouble, contact 800-577-8767
- 2.) Complete your scan and fill out the online Rx form. Be specific in the notes section if you do not want a model or other instructions.
- 3.) Select Pan-Am Dental Lab to receive the file.

